

1. Start Here



Obtain new Therma-Liner™.
Write Baby's name in
Therma-Liner ID box.



Place feeding into
inner bag. Zip closed.



Add LUKEWARM* water to
outer bag up to fill line.
Seal closed.



Place Therma-Liner™ in
empty Penguin cup and
push bag and container
to the bottom.

2. Choose Mode


WARM Feeding

Press 'WARM/OFF'



WARM Frozen Milk


Press 'WARM/OFF'

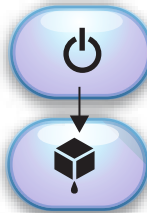

Press 'Frozen' Once

THAW Frozen Milk

Press 'WARM/OFF'

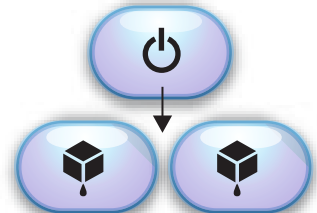

Press 'Frozen' Twice

3. Select Size



Press the Selection button until you
find the best match for container
type, size and volume.

4. Begin Warming

Warming



Ready



When light is red

- Warming or thawing is underway

When light turns green

- Thawing is complete; remove immediately OR
- Feeding is ready and will stay warm for 30 minutes

5. To Finish

Remove feeding from Liner

Press 'WARM/OFF'



Dispose of Liner properly